

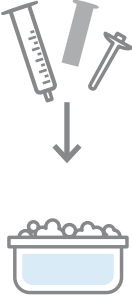

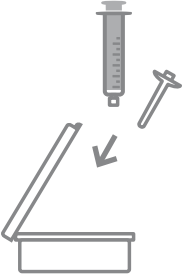



Cleaning instructions: syringe and tip extender

Proper cleaning of the syringe and tip extender after use is important to prevent contamination of the medication. The following information is for advice purposes only. Always follow the instructions of your doctor or pharmacist when using this product. The cleaning of the syringe and tip extender is depicted below. It is not necessary to clean the tip extender when it is not used for administration.

<p>1.</p>  <p>Fill the syringe (with tip extender) with tap water or cooled boiled water.</p>	<p>2.</p>  <p>Flush the syringe (with tip extender). Repeat 1-3 times until all original contents have been cleared.</p>	<p>3.</p>  <p>Separate the syringe barrel, piston (and tip extender). Clean the syringe, piston (and tip extender) in warm water with soap.</p>	<p>4.</p>  <p>Rinse the syringe barrel, piston (and tip extender) with cold tap water and dry everything with a paper towel.</p>	<p>5.</p>  <p>Reassemble the syringe barrel and piston. Store the syringe (and the tip extender) in a clean, dry area.</p>	<p>6.</p>  <p>max. 7 DAYS</p> <p>Discard the syringe (and tip extender) after a maximum of 7 days from first use.</p> <p>The syringe should also be discarded if the piston becomes very stiff to move in the syringe barrel or if the graduated markings become unreadable.</p>
--	---	--	--	---	--