MediSpend™

Patient information leaflet • Packungsbeilage • Notice du médicament • Gebruiksinstructie • Příbalová informace

• Príbalový leták • Φύλλο οδηγιών χρήσης • Foglietto illustrativo • Prospecto • Folheto

EN Instructions for use

MediSpend™ - A lightly viscous liquid with lemon flavor that eases the oral administration of tablets, capsules and other solid dosage forms.

What is MediSpend™?

MediSpend™ is uniquely developed to ease the swallowing of solid dosage forms such as tablets and capsules. MediSpend™ has a pleasant lemon flavor that helps mask medicine taste and has a lightly viscous consistency. MediSpend™ can be used to both aid the administration of intact tablets/capsules or crushed tablets and opened capsules. Consult your health professional and read the patient information leaflet of your medication before doing so.

MediSpend™ is a Class I Medical Device.

MediSpend™ is formulated using ingredients that are commonly used in food to make it suitable for all patients.

Intended use

MediSpend™ can be used to aid with oral intake of different solid dosage forms. These include, but are not limited to:

- Medical pills, tablets, and (liquid) capsules
- Vitamin preparations
- Food supplements
- Other solid dosage forms meant to be swallowed

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